



Pukka-j, Blackwell Farm  
Bishops Cannings  
Devizes, Wiltshire  
SN10 2JZ, UK

Tel: +44 (0) 1380 860044  
Email: [support@pukka-j.com](mailto:support@pukka-j.com)  
Web: [www.pukka-j.com](http://www.pukka-j.com)

## Release Notes

### **Pukka-J DICOM Explorer 6.4.0**

(Released 08/06/2022)

#### **LDAP Authentication**

Added in option to set ldap auth type

#### **JDK 1.4 Support**

Removed some specific support for Java 1.4 that relied on proprietary Sun code that is not in the open jdk.

#### **DICOM Timeout**

Added timeout prop for release response.

#### **DICOM Proxy Release Processes**

Changed the proxy to not close when the complete data PDU arrives.

Added option to allow it.

Changed close process on release request.

#### **DICOM Proxy Property**

Modified the property to make it based on the complete PDU rather than the release as it wasn't clear what it meant.

It was:

`DicomProxyService.CloseOnRelease`

Now:

`DicomProxyService.CloseOnCompletePDU`

#### **DICOM Charset**

Allow the character set to be specified in the DICOM Query to allow study description and patient name etc to be encoded in unicode.

Set the character set on the data message item, not just the command block.

