



Pukka-j, Blackwell Farm  
Bishops Cannings  
Devizes, Wiltshire  
SN10 2JZ, UK

Tel: +44 (0) 1380 860044  
Fax: +44 (0) 1380 860066  
Email: [support@pukka-j.com](mailto:support@pukka-j.com)  
Web: [www.pukka-j.com](http://www.pukka-j.com)

## Release Notes

### **mint.jar 1.23.00**

(Released 04/10/2021)

#### **Workflow Management Node**

New node to be used in conjunction with the workflow management service for handling unsolicited DICOM and HL7 messages to provide imaging and data workflow management.

#### **HL7 Option for Exec Node**

Added a HL7 option to the exec node and created a HL7 exec worker.

#### **JSON Output Node**

Created a JSON file output node to write a JSON file out based on mappings to either HL7 values for DICOM tag values.

#### **HL7 Splitter**

Made the pipe splitter allow HL7 streams to be redirected down multiple channels based upon HL7 mapping rules.

#### **Task Manager CSV Load**

Added line trimming to the CSV file used for the task manager.

#### **File Reader Tag Size**

The default tag sizes were updated.

#### **CMOVE Proxy Node Destination Override**

Added the option to override the destination AET in the CMOVE request in the CMOVE Proxy Node.

#### **Patient ID Manager Node**

Created a new node to allow an alternate patient ID to be looked up by the issuer from the Other Patient IDs Sequence and used as the primary ID by replacing the value in the Patient ID tag.

#### **ID Generator Node**

Created a new node to create a new ID from the hashcode of another tag value.

#### **DICOM File Reader Error Code Handling**

Added an option to handle specific error codes in the DICOM File reader to rename the input file, rather than leaving it in the queue.