



Pukka-j, Blackwell Farm  
Bishops Cannings  
Devizes, Wiltshire  
SN10 2JZ, UK

Tel: +44 (0) 1380 860044  
Email: [support@pukka-j.com](mailto:support@pukka-j.com)  
Web: [www.pukka-j.com](http://www.pukka-j.com)

## Release Notes

### **pjk.jar 1.18.01**

(Released 26/04/2022)

#### **Storage Classes**

Moved storage classes to own class and added reading from package and external file definitions.

Updated build file to include the package storage class definitions.

Removed the 1.2.840.10008.5.1.4.1.1.128, Positron Emission Tomography Curve Storage from the retired list as it is the PET Image store in the main list.

Refactored the use of the defs in the kernel project and deprecated the definitions in the DicomUIDs class.

Removed the package defs of the SOP classes. Just use files to override the defaults.

Created a list of common SOP classes to propose out because by using the now complete list when using a single connection per syntax, there would be too many (>127) according to the standard. The common list is to provide the most commonly seen SOPs and propose them, and any outside this list would trigger a new association.

Removed deprecated constants from DicomUID.