



Pukka-j, Blackwell Farm  
Bishops Cannings  
Devizes, Wiltshire  
SN10 2JZ, UK

Tel: +44 (0) 1380 860044  
Fax: +44 (0) 1380 860066  
Email: [support@pukka-j.com](mailto:support@pukka-j.com)  
Web: [www.pukka-j.com](http://www.pukka-j.com)

## Release Notes

### **pacsoology.jar 1.14.00**

(Released 12/07/2019)

#### **Pixel Normalisation**

Modified the server-side frame capture class to not normalise the pixel data by default but to send it as is and leave the client to manage this. The client can ask the server to do this in the URL call.

#### **Modified Order Upload**

Fixed an issue with the DICOM Map output of the order object.

Added the update order code to the IIS webclient for the add request form.

#### **System Number Generator**

Update the algorithm for generating new system numbers (e.g. patient ID, accession number) to give better control over length and format.

#### **Study Context Query**

Fixed a bug with the query as the table details shouldn't be upper case as that won't work on a Linux OS.